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USER'S GUIDE

Please read these instructions carefully before use, and keep this guide for future reference.

Product description (Fig. 1)

- 1 - ON/OFF switch
- 2 - Battery compartment
- 3 - Slide button to extend / retract comb teeth
- 4 - Comb teeth

How it works

The flea comb effectively eliminates fleas without having to resort to chemical products. It emits an electric charge through the comb teeth that instantly kills fleas and their eggs on contact. The electric charge is so gentle that it is hardly noticeable, and perfectly harmless to your pet. The flea comb is suitable for house pets, such as dogs and cats.

Installing / removing the batteries

Slide the battery compartment cover off. Insert the batteries (AA LR6 x 2), observing the proper polarity (+/-) as indicated on the batteries. Slide the battery cover back on, pressing until the cover snaps into place.

How to use

- 1 - The flea comb is only used on dry pet fur. If necessary, detangle your pet's coat prior to using the comb.
- 2 - To switch the unit on, slide the ON/OFF switch upward. A high-pitched humming sound indicates that the unit is operating.
- 3 - Place the comb at a 45-90° angle and guide it slowly through your pet's coat.
- 4 - The comb's teeth might generate electrical sparks while you are combing your pet. These are painless and harmless to your pet.
- 5 - Always remember to switch the unit off after use, and clean it thoroughly. Always keep the comb's teeth retracted when not in use.
- 6 - Since the flea comb is primarily effective on adult fleas, continue using the comb every day for about 2 weeks to ensure that you eliminate all fleas.
- 7 - To prevent future infestations, we recommend that you use the flea comb once a week as a precaution.

Cleaning & maintenance

- Regular cleaning and good maintenance ensure optimum results and extend the life of your unit.
- Clean the comb thoroughly after each use, especially if it is used on more than one pet, to avoid transmitting fleas from a pet to another.
- To clean the plastic housing, use a soft cloth. To clean the comb teeth, use a cleaning brush.
- Do not use any strong detergents, as they may damage the surface of the unit.
- Never immerse the unit in water and make sure that no liquid gets into the unit.

Technical specifications

Power supply :	2 x 1,5-V LR06 batteries (not included)
Dimensions (L x W x H) :	approx. 170 x 80 x 30 mm
Weight :	approx. 67 g (with batteries)

Cautions for use

- Use the unit only as intended, in accordance with the directions included in this user's guide.
- Using the unit for any other purpose will void your guarantee.
- The unit is not designed for commercial use.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and/or knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children shall not play with the appliance.
- The unit is not suitable for children under the age of 3.
- If you have a pacemaker, artificial limbs or an electronic implant, please consult your physician before using the unit.
- The unit is not recommended for use by persons suffering from epilepsy or heart conditions.
- Do not use the unit on parched, cracked or wounded skin.
- When using the flea comb, avoid direct contact with your pet's eyes and mouth.
- Avoid touching the comb's teeth when in use.
- The comb has rounded pearl-shaped tips on every other tooth. Make sure that no teeth are missing or damaged before you use the unit, as this may cause a short-circuit.
- In the event of a malfunction, do not attempt to repair the unit yourself; this will void your guarantee. Repairs may only be carried out by an authorized service center.
- Keep the unit away from heat and do not expose to direct sunlight.
- Keep away from water and moisture. Should liquid get into the unit, remove the batteries immediately and refrain from using the unit. In that event, contact your retailer or contact NUM'AXES directly.
- Do not use the unit if the plastic housing or comb teeth are damaged, if the unit does not function properly or if it has been dropped or immersed.
- Always switch the unit off when not in use.
- Remove the batteries from the unit when not in use for an extended period. Otherwise, the batteries may start leaking.
- Store the unit in a cool, dry and clean place.

Troubleshooting

Should your product stop working or develop a fault, please read this guide over and make sure that the malfunction is not due to user error or worn out batteries.

If the problem persists, contact your retailer or check out our FAQ section at www.numaxes.com. You may also contact NUM'AXES at **+33 2.38.69.96.27** or by e-mail at export@numaxes.com.

Depending on the extent of the malfunction, you may have to return your product for service and repair.

For repairs, please return the complete product and the proof of purchase (invoice or sales receipt).

Warranty

NUM'AXES guarantees the product against all manufacturing defects for two years after purchase. All postage and packing charges will be the purchaser's sole responsibility.

Warranty conditions

- 1 - The warranty applies only upon submission of a valid proof of purchase (invoice or sales receipt). The warranty is exclusive to the original purchaser.
- 2 - This warranty does not cover the following:
 - battery replacement,
 - direct or indirect transport risks linked to the return of the product to NUM'AXES,
 - product damage resulting from:
 - negligence or fault on part of user (e.g.: bites, breakage, cracking),
 - use contrary to instructions or for other than intended purpose,
 - repairs conducted by unapproved technicians.
 - loss or theft.
- 3 - If the product is found to be defective, NUM'AXES shall either repair or exchange it, as deemed fit by NUM'AXES.
- 4 - NUM'AXES shall not be held liable in any way for the damages resulting from incorrect use or product breakdown.
- 5 - NUM'AXES reserves the right to modify the characteristics of its products with a view to making technical improvements or to complying with new regulations.
- 6 - The information contained in this guide may be subject to amendment without prior notice.
- 7 - Photos and drawings are not contractual.

End-of-life product recycling or disposal

The pictogram affixed on your product indicates that it cannot be disposed of with household waste. You must either bring the device to a collection point for treatment, recovery, recycling and elimination of electronic waste, or return it to your retailer. Safe disposal of electronic waste is an environmentally friendly way to support the preservation of natural resources and the protection of human health.

For further information about collection points for used equipment, you can either contact your local government/ waste management services, or return the product to NUM'AXES.

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EYENIMAL Flea Control NGDIVEY002

FCC Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Any changes or modifications to this equipment not expressly approved by NUM'AXES may cause harmful interference and void the FCC authorization to operate this equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Responsible party:

EYENIMAL INC
World Financial District
60 Broad Street – Suite 3502
New York, NY 10004
sales@eyenimal.com

IC Warning Statements

This digital apparatus complies with Canadian CAN ICES-3(B)/NMB-3(B).
Cet appareil numérique est conforme à la norme CAN ICES-3(B)/NMB-3(B) du Canada.

Z.A.C. des Aulnaies – 745 rue de la Bergeresse – C.S. 30157
45161 OLIVET CEDEX – FRANCE
www.numaxes.com – info@numaxes.com